

User Guide for Andatech AL6000 Pro

ANDATECH AL-6000 Pro Digital Breathalyser (Alcohol Tester / Detector)

Normal Mode Operation

If you attach and use the mouthpiece provided with the unit, you are operating the device in Normal Mode.

1. Instructions

a) Press the Power Switch to turn the unit on. The cumulative test count will be displayed. b) Press the Power Switch one more time and you will see the letters in the display, indicating Normal Mode operation. There will be an automatic countdown from to as the unit warms up.

c) Secure your lips around the mouthpiece and blow sharply and steadily into the unit for at about 5 seconds. You will hear two beeps when the device has finished taking the breath sample. You will see the digit moving left to right in the display as the device is analyzing the sample.

d) After analysis is complete, you will see the blood alcohol concentration (BAC) results displayed.

e) The results will be displayed for approximately 15 seconds, after which the unit will shut itself off.

f) If the device cannot take a successful sample, you will see in the display and the unit will shut itself off. 2. Understanding Your Results (BAC Reading) 0.00% to 0.01% Safe range. Little/no alcohol intake. 0.02% to 0.04% Moderate * range. Increased alcohol intake. 0.05% and over Warning * range. Ability to drive may be impaired. WARN light displayed on unit.

* Consumption of alcohol in ANY amount may impair the ability to operate a motor vehicle. There is no "safe" level for alcohol consumption before driving. Don't drink and drive.

Fast Mode Operation You can operate the unit in Fast Mode without using a mouthpiece. During Fast Mode, the AL6000Pro works as a mere alcohol responder. It means it will only test the presence alcohol but it won't give you the specific blood alcohol concentration. The fast mode is often preferred when the unit is used to measure a large group of people because the test would be less time consuming and hence, more convenient.

1. Instructions

a) Press the Power Switch to turn unit on. The cumulative test count will be displayed.

b) Press the Power Switch one more time and you will see the letters in the display, indicating Fast Mode operation. There will be an automatic countdown from to as the unit warms up.

c) Blow sharply and steadily into the sensor opening near the top of the unit for at least 5 seconds. You will hear two beeps when the device has finished taking the breath sample. You will see the digit moving left to right in the display as the device is analyzing the sample. After analysis is completed, you will see the alcohol concentration range results displayed.

d) In FAST mode, you can immediately do another test when you see on the screen or after the unit has shown the measurement for the previous test.

2. Understanding Your Results (BAC Reading) 0.00% to 0.01% Safe range. Little/no alcohol intake. 0.02% to 0.04% Moderate * range. Increased alcohol intake. 0.05% and over Warning * range. Ability to drive may be impaired. WARN light displayed on unit.

* Consumption of alcohol in ANY amount may impair the ability to operate a motor vehicle. There is no "safe" level for alcohol consumption before driving. Don't drink and drive.